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**Tools Required:**

Tightening Wrench (included inside)  
Slotted Screwdriver

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**Specifications:** Mounts on tub walls from 3 3/4" - 5 3/4" wide

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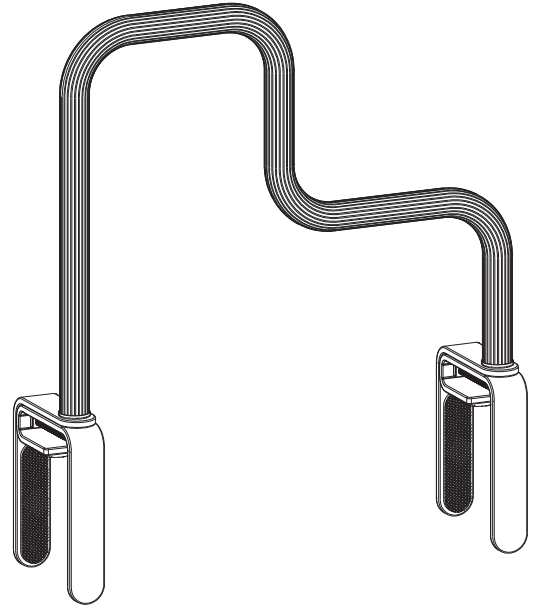
**Dimensions:** 7" low and 15" high grips, 16 1/2" X 15" high, installed

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**WARNING / AVERTISSEMENT / ADVERTENCIA**

TO PREVENT SERIOUS OR FATAL INJURY: Ensure product is assembled, installed, adjusted and used properly. FOR USE ONLY BY PERSONS WEIGHING LESS THAN 250 POUNDS. Failure to follow correct installation procedures can cause the grip to become loose. Check after installation to make sure grip is secure, and check periodically to be sure grip remains secure. **For use only to assist in balance while entering and exiting tub.** Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Do not modify or substitute any components. FOR QUESTIONS OR ASSISTANCE WITH ASSEMBLY OR INSTALLATION CONTACT CSI AT 1-800-882-0116. Consult healthcare professional for advice and instructions specific to user's needs before use.

For additional warranty information, visit [www.moen.com/homecare](http://www.moen.com/homecare)

Meets or exceeds applicable requirements for the Americans with Disabilities Act.

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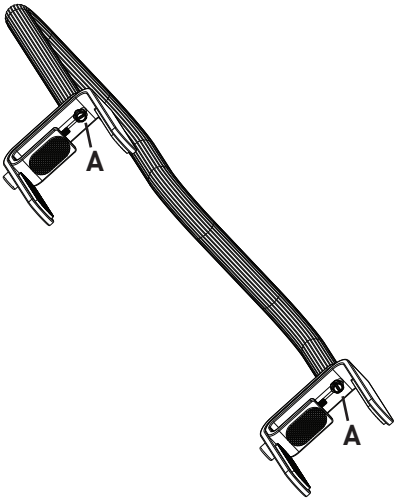
**Cleaning Instructions:** Wipe clean with a soft, damp cloth. Detergents and cleansers may harm the finish of this product.

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<1

**Step 1: Loosen Clamps**

Loosen A and B bolts to allow bar and clamps to be adjusted to your tub.

**Step 2: Adjust Clamp Angle**

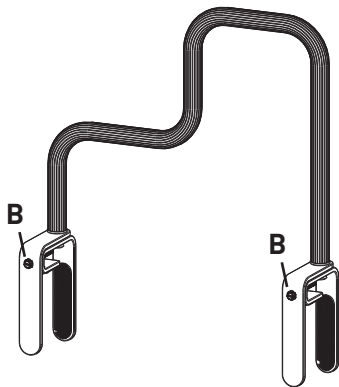
Use the wrench provided to tighten A bolts so that the clamps are angled correctly for your tub.

**Step 3: Adjust Clamp Width**

Position the clamps to fit snugly against the sides and top rim of the tub and then adjust B bolt. Do not tighten yet.

**Step 4: Tighten Clamp to Tub**

Tighten B bolts firmly so that the clamp pads make secure, continuous contact with both sides of the tub wall. Retighten A and B bolts as necessary. **DO NOT OVER-TIGHTEN!**

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