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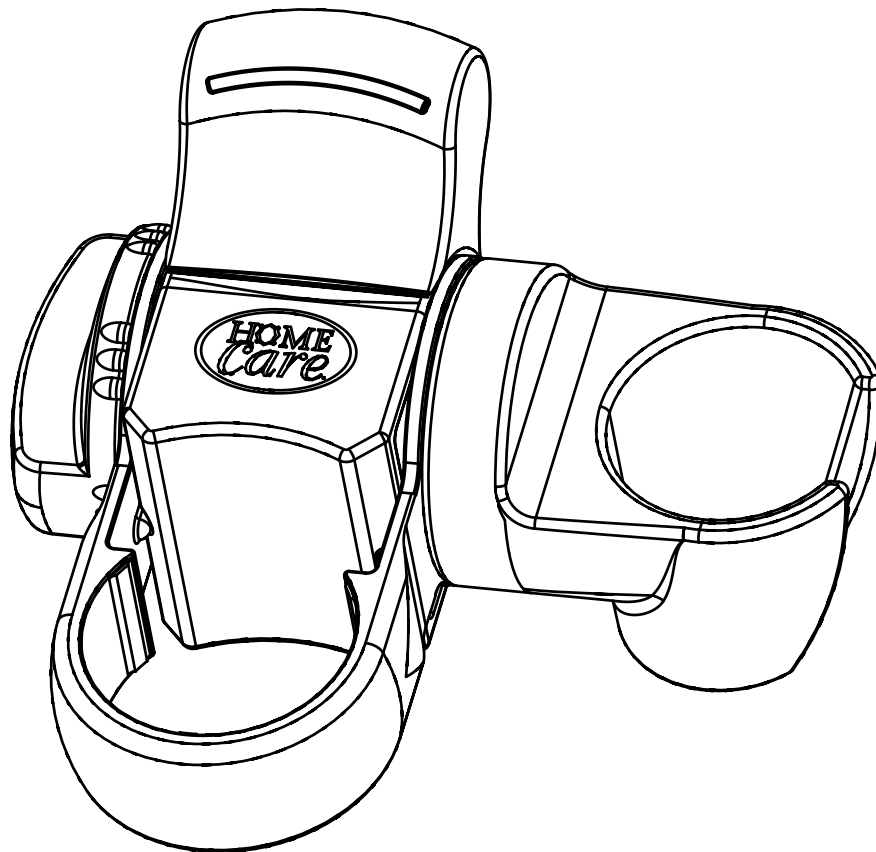
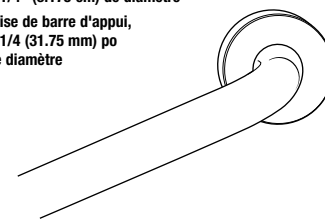
Identify your grab bar style, either 1-1/4" or 1-1/2" diameter. Grab Bar Holder comes pre-assembled with a 1-1/4" grab bar grip for 1-1/4" diameter grab bars. An additional clip is provided for 1-1/2" diameter grab bars.

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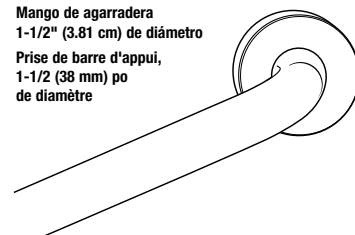
INNOVATIVE DESIGNS
for independent living™

DN8005
GRAB BAR SHOWER HOLDER INSTRUCTIONS
INSTRUCCIONES PARA EL SOSTENEDOR
DE LA DUCHA DE BARRA PARA ASIR
DIRECTIVES POUR SUPPORT DE DOUCHE
DE BARRE D'APPUI

Grab bar Holder comes pre-assembled with 1-1/4" grab bar grip for 1-1/4" diameter grab bars. An additional grip is provided for 1-1/2" Diameter grab bars.

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
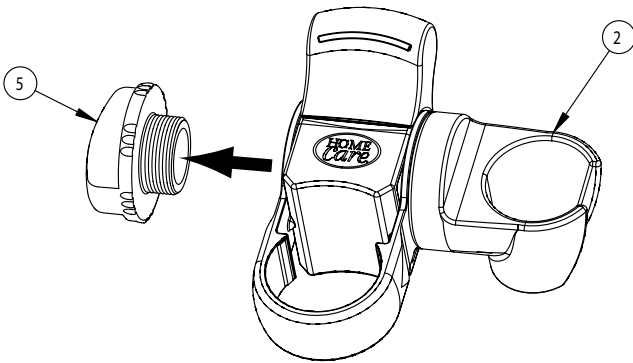
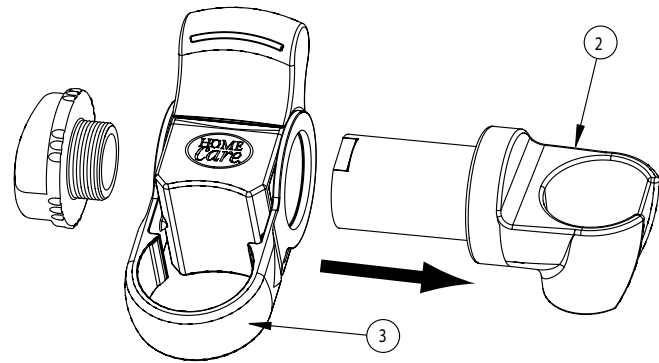
3

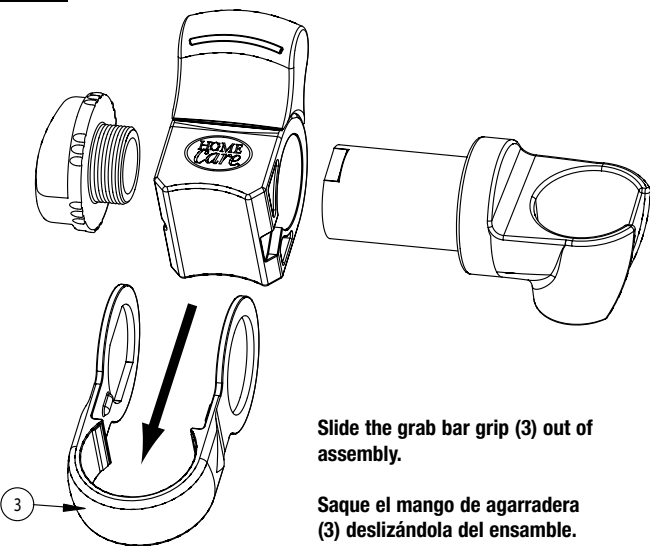Slide the grab bar grip (3) out of assembly.

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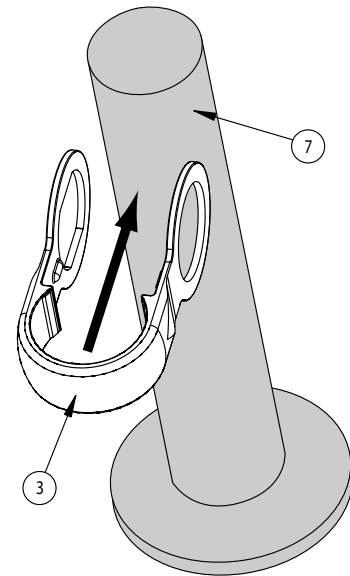
4

Identify the diameter size of your Grab bar. For 1-1/4" Grab bars, use the 1-1/4" Grab bar grip (3) that was pre-installed with your Grab Bar Holder. For 1-1/2" Grab Bars, use the alternate grip provided (6) After selecting the Grab Bar grip, slide it around your Grab bar (7).

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With the bar grip around your grab bar, locate the tab (A) inside the grip and line it up to the triangular cut (B) inside the handle assembly (4), and slide the handle assembly into place.

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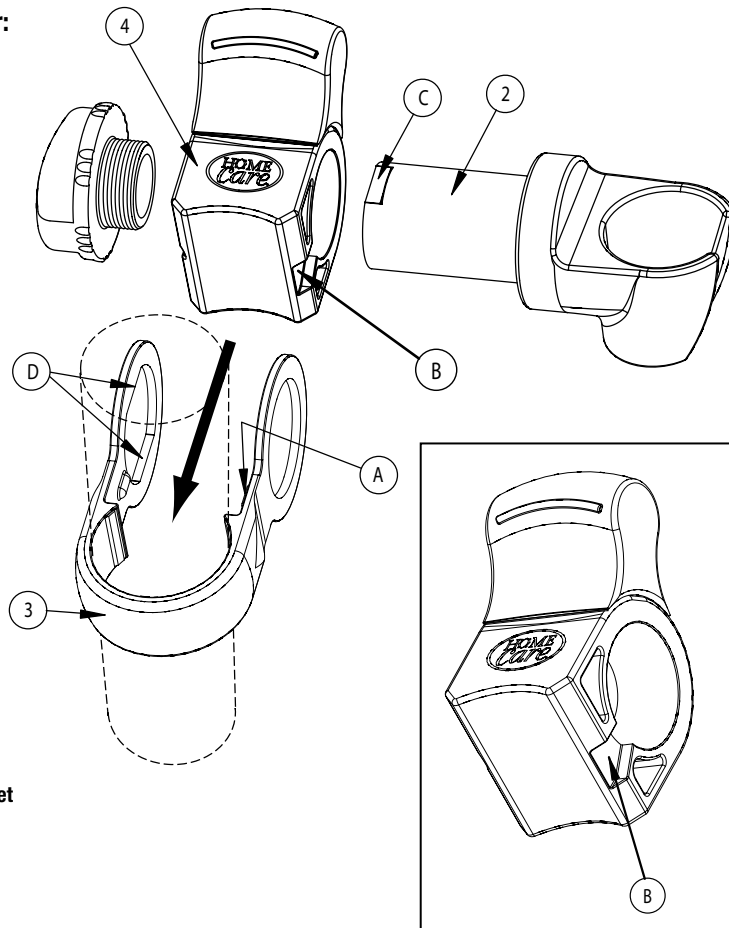
Locate the cradle (2), and align the cradle with notch (C) in up position and pass it through the handle assembly (4) so it lines up with the tabs (D) on the bar grip, and slide it into place.

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
6

Screw cap (5) onto assembly (4). Do not over tighten.

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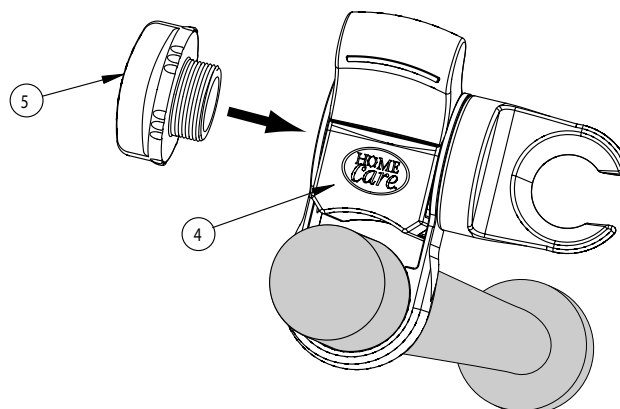7

To position your grab bar shower holder on the grab bar:

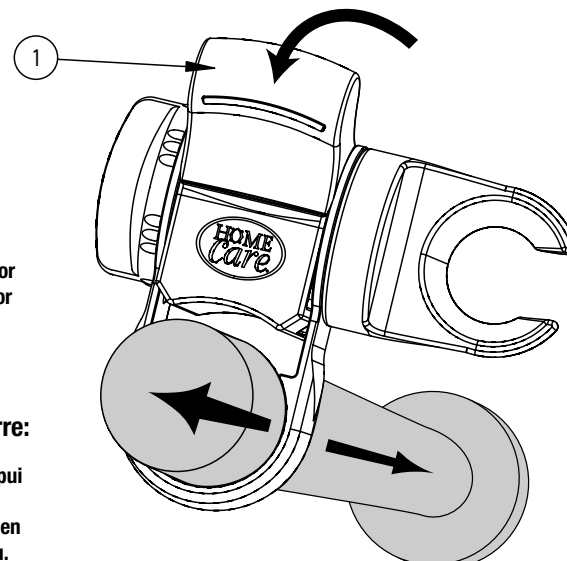
Position the holder along the grab bar. While holding the Grab bar holder in place, push down on the tab (1) of the holder assembly and slide the holder along the grab bar to the desired location. Flip the tab up to lock into place. Rotate the cradle to the desired angle.

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WARNING: Follow these installation instructions carefully. Proper installation is the installer's responsibility. Failure to follow correct installation procedures may cause the product to fail, which could result in damage or injury. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Assemble in accordance with CSI's instructions. Do not make substitutions or modify this product in any way. FOR QUESTIONS CONTACT CSI AT 1-800-882-0116.

LIMITED LIFETIME WARRANTY: If this product fails due to a defect in materials or workmanship at any time during the life of the product, Creative Specialties International, a division of Moen Incorporated (CSI) will replace it free of charge, postage-paid. Simply contact CSI at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY: THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.homecare.moen.com.

The Home Care Grab Bar Shower Holder meets or exceeds the applicable requirements of the following standards / guidelines:

The Americans with Disabilities Act (ADA)

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Creative Specialties is a division of Moen Incorporated. © 2005
Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated.
Home Care and Creative Specialties are registered trademarks of Moen Incorporated.