**Installation Instructions:**

1. Before attaching to a surface, check the suction cups to ensure they are clean from any debris
2. Press the suction balance bar on a clean, smooth, dry surface
  - a. Ensure the surface is flat, clean, dry, smooth and that the bar does not overlap any grout lines
  - b. Not to be used on painted surfaces i.e.drywall.
  - c. Works on 4" x 4" tile
3. While pressing the balance bar to the surface, press down each locking lever one at a time
4. Before each use, test the balance bar for secure placement on the surface
5. To remove, unlatch both locking levers and slide fingertip under rubber seal at each end to release suction.

**Caution:**

1. The suction balance bar is for balance assist only and not for body weight leverage. Visit [www.moen.com](http://www.moen.com) for more permanent grab bar solutions.
2. Check the balance bar to make sure it is securely attached before each use. Reattach if necessary.

**Maintenance:**

1. Keep the suction cups clear of oil and dust
2. Clean the suction cups with standard household cleaners
3. Keep suction cups away from sunlight for long periods of time
4. Periodically, check the suction cups for deterioration. Replace if necessary

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### WARNING:

Consult a healthcare professional for advice and instructions specific to your needs before use. Contact CSI immediately at 1-800-289-6636 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-289-6636.

### LIMITED LIFETIME WARRANTY:

If this product fails due to materials or workmanship at any time during the life of the product, Creative Specialties International (CSI), a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact CSI at 1-800-289-6636 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn.

TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

**LIMITATION OF LIABILITY:** THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit [www.moen.com](http://www.moen.com)

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Creative Specialties is a division of Moen Incorporated. © 2011
Creative Specialties are a registered trademarks of Moen Incorporated.