# **Home Care Technical Specification**

## **Adjustable Transfer Bench**

**DN8065** 

## **Stock Numbers:**

DN8065 – Adjustable
 Transfer Bench
 Adjustable seat height from 17"
 to 21"

### **Materials:**

Adjustable Transfer Bench is made from high impact polypropylene. Legs are polished anodized aluminum. Feet are gray rubber.

ASSEMBLY TESTED TO CONFIRM ADA COMPLIANCE




#### **WARNING**

TO PREVENT SERIOUS OR FATAL INJURY: Insure product is assembled, adjusted and used properly. FOR USE ONLY BY PERSONS WEIGHTING LESS THAN 300 POUNDS. Failure to follow correct assembly and/or adjustment procedures can make the seat unstable. ALL FOUR LEGS MUST BE LOCKED IN PLACE AT THE SAME HEIGHT. DO NOT jump on, stand on or tilt. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Do not modify or substitute any components. FOR QUESTIONS OR ASSISTANCE WITH ASSEMBLY CONTACT CSI AT 1-800-882-0116. Consult healthcare professional for advice and instructions specific to user's needs before use.



## Dimensions:

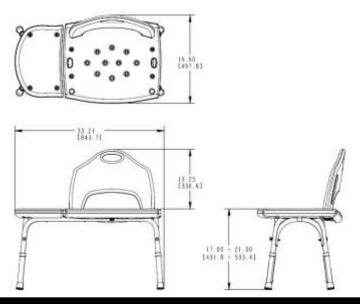

## **Installation Instructions:**

- 1. Position legs. Place leg "1" into grove "1" of the plastic seat panel so that the number "1" marks are next to each other. Then place leg "2" into groove "2", over leg "1" so that the number "2" marks are next to each other.
- 2. Attach legs. Insert a 1 ½" screw through a washer and the center hole of the legs and seat. Tighten several turns. Insert remaining four screws through washers into the four leg holes. Tighten all five screws fully. Test seat. Retighten as needed.
- Adjust leg height. Push in button on leg, slide leg to desired height and allow button to pop out and lock leg in place. Repeat for each leg, making sure all four are adjusted to the same height.

## **Cleaning Instructions:**

Clean only with a soft, damp cloth. Do not use commercial or abrasive cleaners.

Stock Number:\_\_\_\_\_Adjustable

Transfer Bench By Creative Specialties

25300 AL MOEN DRIVE • NORTH OLMSTED, OH 44070 CORPORATE HEADQUARTERS: (800) 321-8809 • FAX: (800) 848-6636 INTERNATIONAL: (440) 962-2000 • FAX: (440) 962-2726 CUSTOMER SERVICE (USA): 800-882-0116 • FAX: (888) 379-2720 WWW.MOEN.COM

©Copyright Creative Specialties International